DE Beheizbare Augenmaske
GB Heated Eye Mask

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Instruction Manual

Modell: EM623

CE

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www.thermrup.de

Contents

1.	Important instructions.....	6
2.	Included in delivery	7
3.	Product features.....	7
4.	Operating.....	7
5.	Cleaning and maintenance.....	8
6.	Disposal	8
7.	Technical data.....	9
8.	Warranty.....	9

1. Important instructions

- If you wear contact lenses, please remove them before using the product.
- Do not use the device on children, disabled or sleeping persons, or heat-insensitive persons who cannot react to overheating.
- Keep the temperature controller and the cabling as well as the supplied power supply away from moisture and liquids!
- If the eye mask is not used for a longer period of time, the power supply should be disconnected.
- The product must not be used by the following groups of persons: babies, deep sleepers, physically or mentally impaired persons, unconscious persons who are not aware of the temperature.
- The product may be used by children and by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the product in a safe way and understand the hazards involved.
- **Stop using the eye mask if it becomes uncomfortably hot.**

2 . Package Contents

Please first check the completeness of the device and inspect the product for damage. If the device is damaged, do not operate it and contact your dealer or the service department.

The scope of delivery includes:

- 1 x eye mask
- 1 x power supply
- 1 x temperature controller
- 1 x instruction manual
- 1 x storage bag
- 6 x cotton pads

3. Product features

The Thermrup heated eye mask with far infrared heat, relaxes dry and tired eyes by the delivered heat. The temperature of the eye mask can be adjusted individually at any time by the included temperature controller. The special rubber heating element has a large heating surface and provides soothing heat. The Thermrup eye mask is waterproof, so the mask can be used dry as well as moistened.

Temperature sensor

4. Operation

4.1 Commissioning

Before use:

- Connect the plug of the eye mask to the socket of the supplied temperature controller.
- Plug the USB connector of the temperature controller into the supplied USB adapter. (If necessary, a separate power bank can also be used as a power supply).
- Plug the USB adapter into the socket (AC100—240V)

To turn on the heating function, press the "ON" button briefly until the display shows the temperature "45".

4.2 Operation

- Now you can set the desired surface temperature of the eye mask: "40", "45" or "50" (3-level temperature settings)
- Press the button plus "+" to increase the temperature or minus "-" to decrease the temperature.
- Then put the eye mask to your eyes and fasten the mask around your head with the connected Velcro strap.
- If you want to turn off the heating function, press and hold the power button for about 1 second until the indicator shows a dot ".".
- After 13 minutes of continuous operation, the heating function will automatically turn off. If you wish to continue using the device, press the On/Off button again to resume using the eye mask according to your individual temperature setting.

Tips for use:

- When starting the heating function, the lowest temperature should be set first (40°C). If you want a drier heating of the mask, set the temperature between 40°C and 45°C.

- If you want the mask to be warmed up with moisture, set the temperature between 45°C and 50°C. The eye mask is waterproof, so you can put e.g. wet cotton pads on your eyes and enjoy the warmth of the Thermrup eye mask at the same time.

5. Cleaning and maintenance

- Before each cleaning, disconnect the temperature controller or the power supply from the eye mask.
- Never use aggressive cleaning agents or strong brushes.
- The eye mask can be washed in the washing machine.
- The eye mask must not be dried in a dryer or ironed.
- Wash the eye mask only if absolutely necessary. Too frequent washing may cause wear marks.
- After washing, do not use the eye mask until it is completely dry.

6. Disposal

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

7. Technical data

Model : EM623
Power supply
Heating pad : 5V / Max. 5W
Power supply Input : 100-240V ~ 50 / 60Hz
Output : 5V; 2000mA
Dimensions : approx. 22x9 cm

8. Warranty

We provide a 2-year warranty from the date of purchase for material and manufacturing defects of the product.

The warranty does not apply:

- In the case of damage caused by improper use.
- For wear parts.
- Customer's own fault.
- Once the unit has been opened by an unauthorized workshop.

For the assertion of a warranty within the guarantee period, the customer must prove the purchase.

In case of complaints please contact our service under the following contact:

E-Mail: service@zewox.de

Zewox GmbH

Oststr.159

47057 Duisburg

Germany

www.thermrup.com